

55
YEARS

Thomashilfen

PIONEER
SINCE 1969

NEW

Tested according
to the new safety
standard R129
(Seat & Accessories)

—
including side impact
protection!



100 - 150 cm
39.3 - 59.1"

15 - 50 kg
33 - 110.2 lbs

Individual Fit: Perfect adjustment through
flexible side wings and headrest

Axkid NEXTKID Reha


Thomashilfen



Axkid NEXTKID Reha

- The side wings can be adjusted independently, e.g., to ensure an optimal seating position for scoliosis or spasms.
- A real lightweight with just 8 kg / 17.6 lbs – can be used up to a body height of 150 cm / 59.1" and a weight of 50 kg / 110.2 lbs.
- The extensive range of accessories, such as a swivel base, makes everyday use easier, especially when transferring heavier and larger children.
- The backrest and 8-position adjustable headrest ensure optimal adjustment, regardless of the child's age or size.



Thomashilfen has been supplying special children with rehab car seats for over 35 years. Today, we also use our expertise to develop our own rehab car seats. They incorporate all our many years of experience in meeting the needs of children and parents.

Gunnar Thomas, owner








Pioneer

REHAB CAR SEATS


since
1988

Thomashilfen has the world's largest range of rehab car seats and offers the right support for all ages and disabilities.

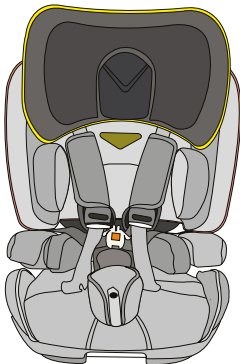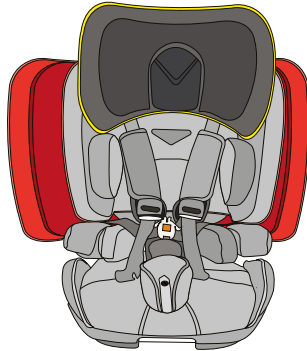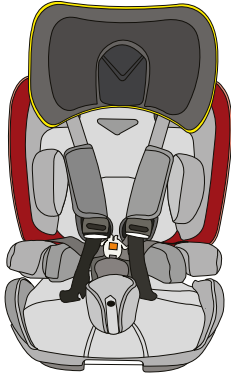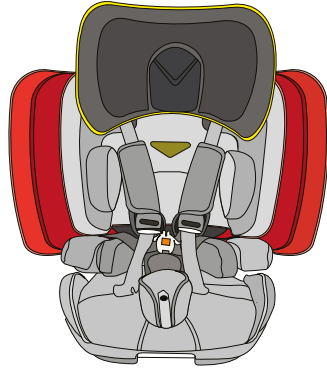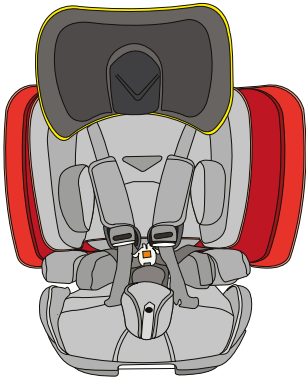

The individually extendable side wings and the separately adjustable headrest adapt perfectly to the individual needs of the disabled child.

Application example

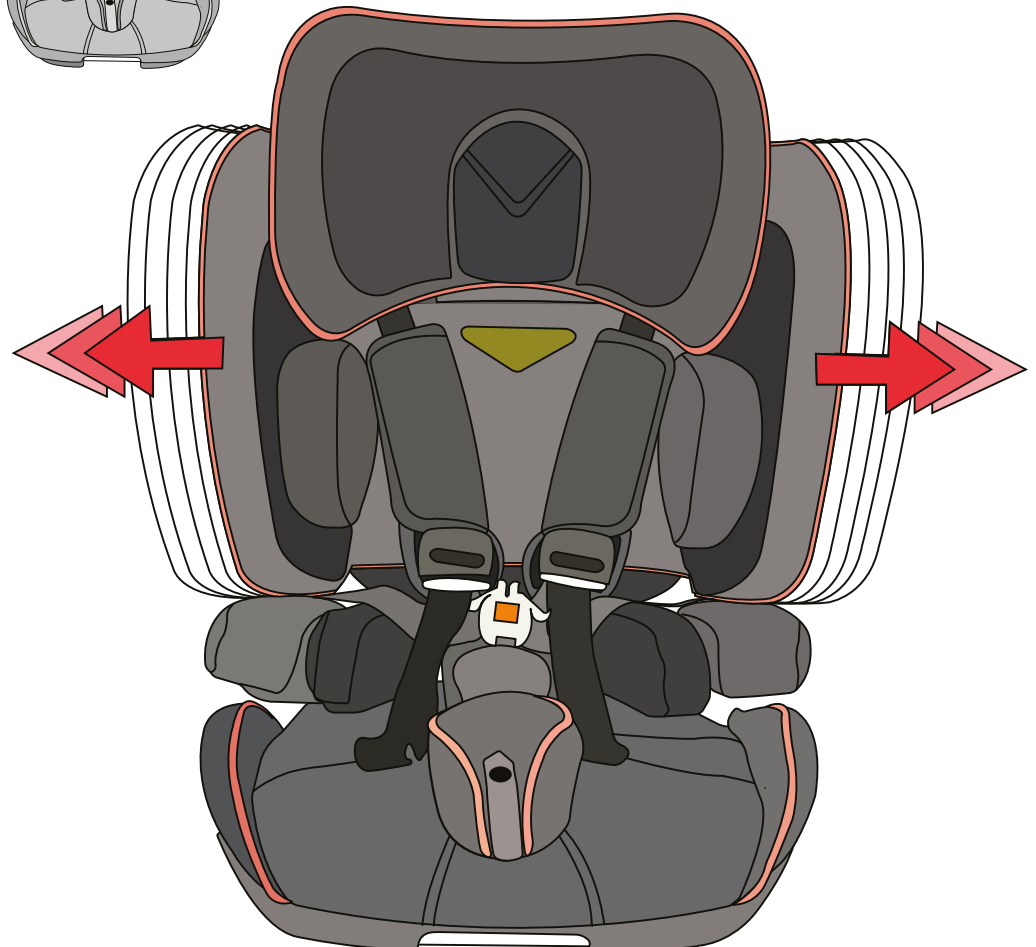

BASIC REHAB EQUIPMENT INCLUDED



The basic rehab configuration can optionally be equipped with a suitable swivel base and tilting base.



The **SWIVEL BASE** and **TILTING BASE** can be supplemented with a wide range of accessories as required. For easy handling, the release lever is located on the side of the swivel base.

HARNESS GUIDE

optimizes harness routing and ensures optimal positioning of the safety belt on the pelvis and prevents the belt from slipping off the shoulder or disturbing the neck.

CROTCH PAD

reduces the pressure of the harness buckle on the sensitive abdominal area.



PADDING

The integrated ventilation system reduces heat build-up and sweating.



GMFCS
Level
III-V

100 - 150 cm
39.3" - 59"
Body size
15 - 50 kg
33 - 110.2 lbs
User weight

HEADREST

8-position height adjustable – easily and quickly adjustable, grows step by step with the child. The headrest can be adjusted separately from the side wings.



ISIP SIDE IMPACT PROTECTION (Integrated Side Impact Protection)

PRECISE ADJUSTMENT

for improved upper body posture: large velour upholstery enables precise attachment of lateral trunk support.



The **SIDE WINGS** can be adjusted independently in four stages. Suitable for various disabilities and flexible for every body size and shape.



ISOFIX

Quick and easy installation in just a few seconds. Safety indicator in green/red for correct anchoring. The ISOFIX bar automatically folds out, making installation even easier.



Axkid NEXTKID Reha

► Functions (selection)




LATERAL TRUNK SUPPORT
improve upper body support and optimally support by attaching precisely to the velour padding.



The **SIDE WINGS** are individually adjustable, adapt to growth, and ensure an optimal seating position.



HEADREST
with soft velcro surface allows the attachment of additional accessories for head support.

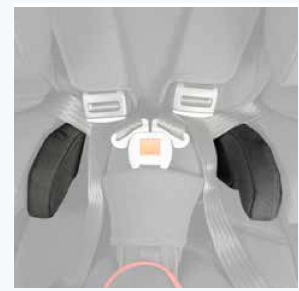


LATERAL SEAT UPHOLSTERY
provides good support especially for slim children.



- The innovative high-tech base expands the car seat with a variety of functions, such as a swivel base, a tilt base, or a combination of both. This flexibility enables easy adjustment and increases comfort and safety for the child significantly.

HIGH-TECH-BASE COMBINATION

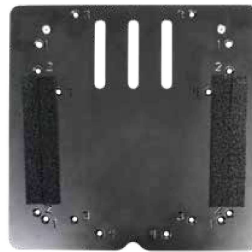

ADAPTER PLATE -
is ALWAYS required
for the following
accessories




SWIVEL BASE
makes transfer easier,
conveniently accessible
release lever can be ordered
for right or left rotation.




TILTING BASE
allows seat tilting in 4 stages
(2.5°/5°/7.5°/10°) for a
relaxed resting position.



**TILTING BASE AND
SWIVEL BASE**
combined for the best
possible handling
and comfort

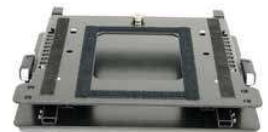

**FOOTREST SUPPORT
FITTING** with or
without **SEATFIX**

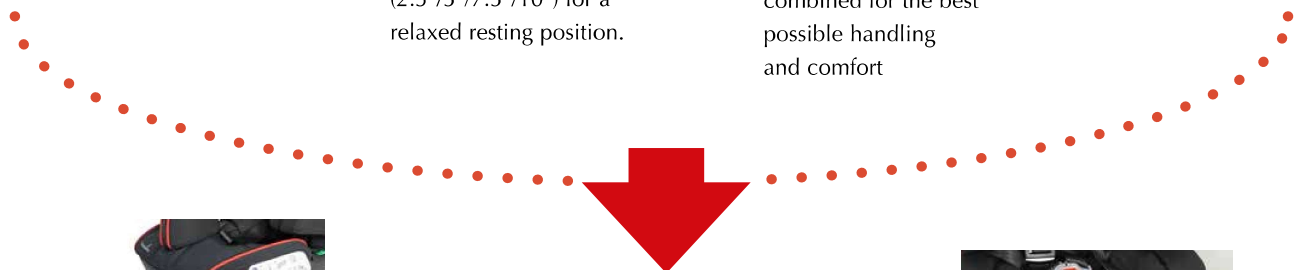

FOOTREST
ensures a stable seating
position, allowing the
child to brace their
feet firmly.




**SUPPORT FOOT,
SEATFIX-ADAPTER OR
STABILIZING BOW**
ensure stable support
of the seat in the car.



ABDUCTION BLOCK
(adjustable) improves leg
control and prevents the legs
from crossing.


► Accessories (selection)

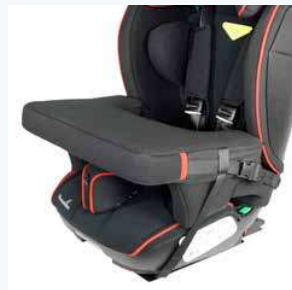

TABLE, ANTERIOR SUPPORT
promotes the child's upright and stable posture, gives the child all-round support and security.



Seat wedge (below)
allows a +10° seat tilt.

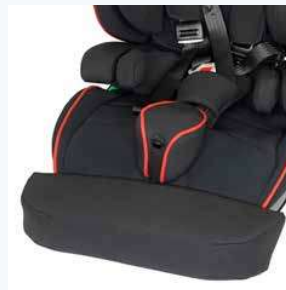

SEAT DEPTH EXTENSION
offers more leg support and individual adjustment for tall children.

A complete overview of accessories can be found on the order form on the internet:
www.thomashilfen.com

Dimensions & weights	Axkid NEXTKID Reha
Seat depth (behind crotch strap)	23 cm / 9.1"
Seat depth	38 cm / 15.0"
Seat width	22 cm - 29 cm / 8.7 - 11.4"
Back height (seat inside)	75 cm - 29.5"
Shoulder height (harness guide)	33 - 51 cm - 11.8 - 20.1"
Shoulder width	36 - 42 cm / 13.8"
Seat tilt (base model)	2,5° / 5° / 7,5° / 10°
Overall dimensions (w x h x d):	44 x 62 x 48 cm / 17.3 x 24.4 x 18.9"
Weight of seat	8,5 kg / 18.7"
User height	100 - 150 cm / 39.4 - 59.1"
Max user weight	50 kg / 110.2 lbs

Standard features:

Seat and backrest with integrated 8-position height-adjustable headrest; integrated 5-point belt with central adjuster; height-adjustable shoulder pads with Axkid belt pad system incl. small crotch pad; removable trunk and seat side pads; 4-stage adjustable side wings, incl. integrated ISOFIX connectors, removable covers.

Watch the video:
How quickly and easily the Axkid NEXTKID Reha is installed in the car.



Member of the International Support Association for the Rehabilitation of Children and Youths e.V.

www.rehakind.com



www.kinderreha24.de

Rehab car seats from Thomashilfen: The right solution for every journey!



Reboarder Axkid ONE 2 Reha

61 - 125 cm
24.0" - 49.2"



AVOVA Sperling Reha

76 - 135 cm
29.9" - 53.1"



Axkid NEXTKID Reha

100 - 150 cm
39.4" - 59.1"



Hercules Small

100 - 150 cm
39.4" - 59.1"



Hercules Prime

130 - 180 cm
51.2" - 70.9"

You can find the entire range of rehab car seats at www.thomashilfen.com



Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG · Walkmühlenstraße 1 · 27432 Bremervörde · Germany
Phone: +49 (0)4761 886-63 or -68 · Fax: +49 (0)4761 886-19 · www.thomashilfen.com