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HPVM001

FallSafe valmat

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FallSafe Fall mat

Intended user:

This medical device is intended for use by medical professionals.

Intended purpose:

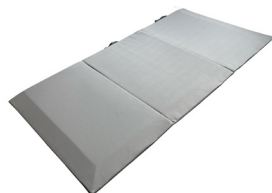
The fall mat helps to minimize injury in the event of a fall from the bed.

Indications for use:

Patients/residents with an increased risk of falling or rolling out of bed with possible injury as a result.

Contraindications:

Resident/patient with a parkinsonian gait pattern.

REF HPVM001

Composition:

Top: EVA-foam

Artificial leather (75% Polyether sulfone, 25% Cotton)

Bottom: 100% Polyethersulfone

Coating: 100% PVC

Explanation of symbols

Manufacturer

Date of manufacture/
Country of origin

Medical Device

Catalogue number

Batch code

Consult instructions for use

Keep dry

Keep away from sunlight

Do not use if package
is damaged

Product description:

A fall mat is used for patients/residents with an increased risk of falling or rolling out of bed resulting in possible injury.

Packaging:

This product is individually packaged.

Storage and handling:

This product was designed to be used indoors. The product can be stored at room temperature. Avoid excessive moisture or high humidity which can damage the product.

Fold the mat when not in use. When the mat is folded, it is only 9 cm thick. DO NOT store the mat on the slanted edge as this may cause the edge to roll up creating a fall hazard

DISCLAIMER! Due to the random nature of falling, Human Protection B.V. makes no guarantees, express or implied, that the user will be protected from injury.

Before applying:

1. Make sure that "This side down" label is on the bottom.
2. Place the mat lengthwise, wit straight (non-bevelled) paralel of the side of the resident's bed.

Cleaning instructions

Clean with lukewarm water, neutral soap and a soft cloth. If necessary, the fall mat can be cleaned with a 70% alcohol solution or 250 PPM chlorine. Protect your hands and clothes when using alcohol or chlorine.

Disposal

Clean with a liquid disinfectant to remove dirt before disposing of the product. Dispose of the product according to institutional policy.

TAKE NOTE! Just as a patient's behaviour is not 100% predictable, no product is 100% foolproof. Patient safety requires regular reassessment and monitoring of hte product and situation. A product that worked before may miss its purpose if the patient's mental of physical health condition changes. Consult the appropriate medical authority if you have any questions regarding patient safety.

Additional warnings:

Inform the resident and staff to be careful when stepping on or off the mat to avoid tripping. Follow your facility's procedures for cleaning up spills under or around the mat. This increases the risk of slipping/falling. If the mat is used in combination with a low-low bed, make sure that there is sufficient distance between the underside of the bed and the mat to prevent the mat from constricting. This could cause damages.

Inform the manufacturer if an incident occurs with the FallSafe fall mat.

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