

## NAGGL mini home edition, Presales information

NAGGL



If you have problems reading this document,  
please download: [www.naggl.nl/naggl-mini](http://www.naggl.nl/naggl-mini)

## Intended use

The NAGGL mobility aid is a class 1 medical device, primarily designed for children with severe neurological diseases and their consequences for the body's motor function. The mobility aid supports the pelvis and the trunk so that active standing is possible first and then active walking with defined compensation of the body motor restriction.

It is intended to use with adult supervision. The NAGGL Mini home edition is intended for indoor use. The product is useable for children who weigh up to 50 kg.

The main Contraindications for use are unstable or irritated wounds or sores on the breast or on the pelvis, inadequate function of the cardio-pulmonal system or severe hyperactivity.

## Maintenance instructions

The NAGGL mini mobility aid can be cleaned with water and a mild cleaning agent (hand soap /dishwashing liquid). After washing, the mobility aid must be dried. The breast cushion the saddle can be disinfected with mild disinfection solutions if necessary.

## Transport and Storage

For transport and storage, you can remove the front axis by completely removing the clamping screws. This results in two components which can be carried easily. You can also remove the lower clamping screw from the push bar. The push bar can then be turned over thus considerably reducing the length of the device.

Size and weight

The size of the NAGGL mini must be adjusted by a professional, this means that the measurements given are estimates of the devices size when adjusted to any user.

Size for use	
Width	+/- 725 mm
Length	+/- 1200 mm
Height	+/- 850 mm
Weight	7,4 kg

Size for storage	
Width	+/- 725 mm
Length	+/- 900 mm
Height	+/- 700 mm
Weight	7,4 kg

Remarks

---

---

---

---

---

---

---

---

---

---

---



For further Information please use the link:

[www.naggl.nl](http://www.naggl.nl)



or contact:

Conixus Benelux B.V.

Edisonweg 41 A9

4382 NV Vlissingen

Mail: [info@naggl.nl](mailto:info@naggl.nl)

