



alfa



GENERAL INFORMATION

The **ALFA special needs stroller** is dedicated to children at a height of **100 – 150 cm** and a maximum weight of **55 kg**.

Its extensive and solid construction, consisting of a frame, seat and separate backrest, makes it extremely durable and significantly facilitates both packing the transport of the device.

Thanks to the "I grow with you" function Alfa allows you to smoothly extend the seat depth (**from 30 up to 40 cm**) and the backrest height (**from 59 up to 74 cm**).

Combined with the possibility of quick backrest recline, tilt, adjustable footrest and adjustable pads, we obtain a comprehensive product that ensures maximum comfort, safety and functionality for many years.

Last but not least, **ALFA** also boasts a modern design, which makes it even more universal and attractive.

THIS IS A MEDICAL DEVICE.

USE IT IN ACCORDANCE WITH THE INSTRUCTIONS FOR USE OR THE LABEL.

INDICATIONS

- paralysis and paresis of lower limbs various origins
- paralysis and paresis of the trunk
- weakening of postural muscles responsible for maintaining the correct body position
- cerebral palsy
- myelomeningocele
- muscular dystrophies

CONTRAINDICATIONS

In case of any contraindications to sitting position &/or passive verticalization stroller adjustments should be consulted with physiotherapist or physician-in-charge.



TILT IN SPACE SEAT



RECLINING BACKREST



ADJUSTABLE FOOTREST



LYING POSITION

CONSTRUCTION

The structure of the **ALFA** special needs stroller consists of two basic elements: seat frame and backrest frame. The backrest is fixed to the seat with the help of a simple locking mechanism (check the manual for details).

Before placing your child in the stroller, make sure that the backrest locking mechanism is properly locked on both sides of the stroller; to do so grab the backrest and gently shake it. A properly installed backrest will remain in its position.



MODIFYING PARAMETERS

With the help of selected accessories and a few simple moves we can tailor the **ALFA** special needs stroller to the needs of children whose height and weight are within the parameters assigned to the model (**height 100 – 150 cm, weight up to 55 kg**).

The stroller has an adjustable pelvic and lateral pads **(1)**, height adjustable headrest **(2)**, height and recline adjustable backrest **(3)**, tilted seat with depth adjustment **(4)** and adjustable footrest **(5)**.



EQUIPMENT




SIDE PADS

ADJUSTABLE LATERAL
AND PELVIC PADS

SAFETY BARRIER



ABDUCTION BELT

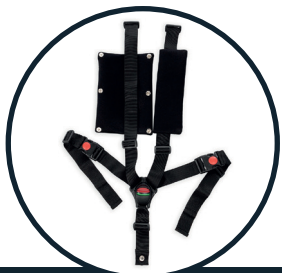

FIVE-POINT BELT

ADJUSTABLE
ABDUCTOR (WEDGE)ANGLE ADJUSTABLE
THERAPEUTIC TABLECANOPY WITH
WINDOW

HEADREST



BACKPACK



SHOPPING BASKET



SUN UMBRELLA



MOSQUITO NET



RAIN COVER



SLEEPING BAG

TECHNICAL PARAMETERS OF THE STROLLER

seat width: 27 – 32 cm*, 41 cm**

effective seat depth: 30 – 40 cm

backrest height: 59 – 74 cm

backrest width: 27 – 32 cm*, 40 cm**

distance between footplate and seat: 30 – 40 cm, 8 positions

*values are given with pads or **) without pads

total weight: 26 kg

stroller length: 100 – 145 cm

stroller height: 107 – 128 cm

stroller width: 66 cm

IMPORTANT NOTE! The choice of a medical device appropriate for a specific disease entity and condition of the patient must be made with the help of a specialist doctor or physiotherapist. Before use, read the operating manual and the label or consult your doctor, as this medical device may not be suitable for you.