

Seating

Standing

Sleeping

Therapy

Standz by Jenx

A versatile standing system offering prone or supine standing that accommodates abducted or neutral leg positioning for children between 1 - 9 years



Always Supportive...

Standz by Jenx

Inspired by the clinical benefits of abducted standing and created alongside therapists and carers all over the world ensures the perfect combination of clinical positioning with simplicity of use. Standz offers prone or supine standing and easily accommodates either abducted or neutral leg positioning for children between the ages of 1-9 years.

Leg length discrepancy is easily accommodated by the individual controls for knee and foot positioning, and all adjustments are tool free to enable quick and effective clinical postural adjustments. With an extensive range of accessories, Standz can accommodate children with a wide range of postural requirements.



Showing Standz in Prone

INNOVATIVE FEATURE

Safety Lock

Built in safety lock for frame angle adjustment – ideal for those busy environments where there is potential for accidental activation of an unlocked mechanism.

INNOVATIVE FEATURE

Tray Holder (optional accessory)

This clever accessory offers convenient storage when the **Tray** is not in use.

INNOVATIVE FEATURE

Angle Gauges

Built in angle gauges for frame angle and individual leg abduction - easy to read gauges enable precise positioning and allow for the correct position to be repeatedly achieved.

INNOVATIVE FEATURE

Individual Leg Adjustment

Adjustment for both knee and footplate height – ideal for accommodating leg length discrepancy. This tool free adjustment can be made in seconds and securely clamps into place using our clever clamping marquee design.

INNOVATIVE FEATURE

Individual Leg Abduction

Standz offers the ability to independently adjust either leg from 0° – 30° of abduction depending on the child's clinical needs. Simply adjusted through a wing knob, this action is quick and simple to adjust for each standing session. An incorporated angle gauge makes it easy to see the degree of abduction for each leg.

INNOVATIVE FEATURE


Antimicrobial Covers

As standing aids the digestive system and gravity can take it's course, our covers have been designed to wipe clean, they also contain an antimicrobial agent to aid infection control.

INNOVATIVE FEATURE

Tool Free Adjustments

No more hunting for the right sized allen key, cleverly designed wing knobs offer a comfortable grip to enable effective clamping with minimal effort – meaning growth adjustments have never been easier.

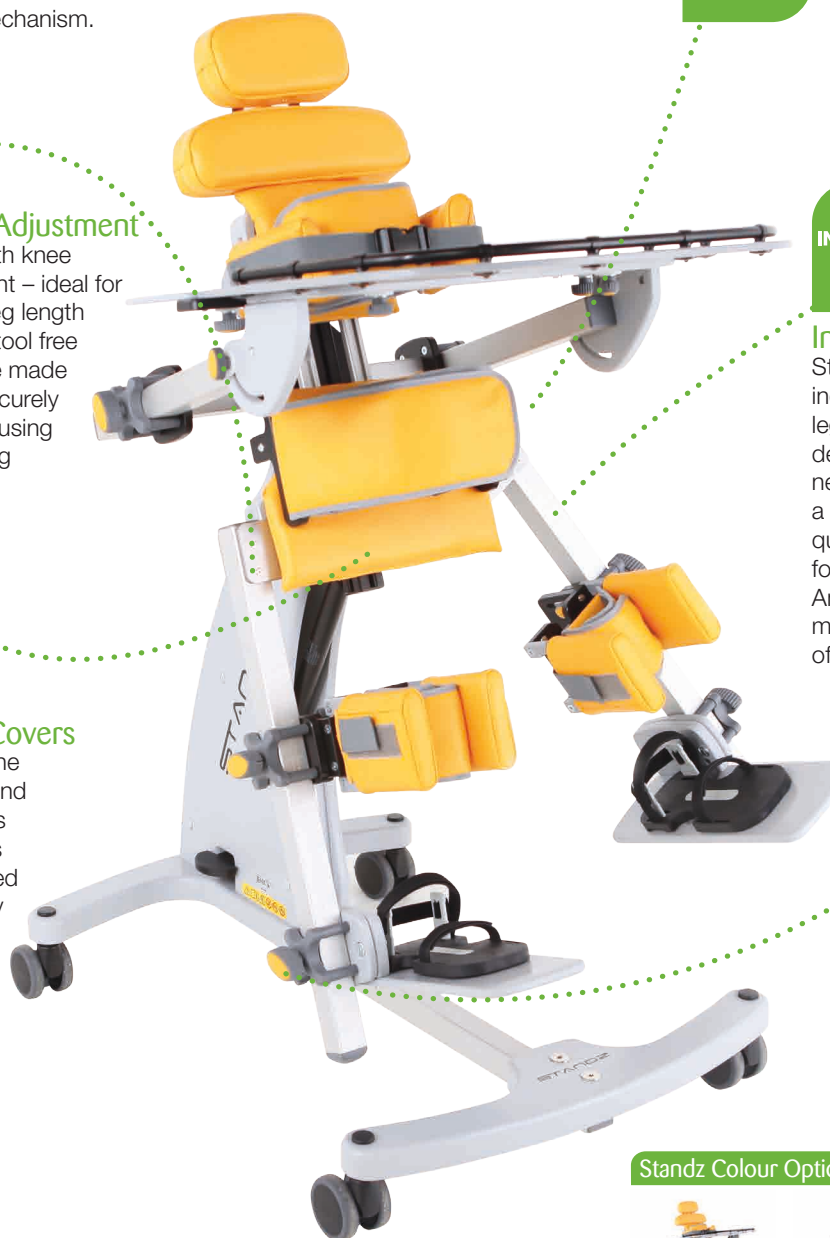

Standz Colour Options



Always Configurable Accessories

As standard each base comes with:

- 4 lockable swivel castors
- Tilt mechanism with foot pedal release and safety lock
- 2 leg posts clearly marked with numbers for easy positioning of leg supports
- 0° – 30° independent abduction movement of leg posts
- Individual footplates with plantar and dorsiflexion movement
- Angle gauge to read frame angle – frame angle range of 5° – 90°
- Pelvic support board and cover pad in a choice of 4 colours
- Trunk support board and cover pad in choice of 4 colours

Flat Pad Head Support

A simple yet effective head support pad for use in supine for children with reasonable head control. It can also be used in prone to extend the height of the chest pad.

Oval or Moulded PU Head Support

Simple headrest for children who require minimal head support. Comes mounted on a flexible fixing bracket.

Multigrip™ Head Support

The original and innovative **Jenx Multigrip™ Head Support** is available in 3 sizes for all Jenx seating systems. Provides adjustments for height, depth and angle.

Flat Shoulder Support

A simple yet effective wide pad for supporting the shoulders when using Standz in a supine position.

Rotation Control Strap

A wide support belt designed to be used to position the pelvis, trunk or both together. Threaded through the support fixings the strap can be fastened securely around the child and then be pulled to facilitate extension or rotation.

Lateral Supports with Strap

Firm lateral support pads offer stability to the trunk, pelvis or both together, and the incorporated support strap can be used to facilitate extension or rotation.

Pelvic Support Board

For use in prone positioning only, the **Pelvic Support Board** offers a full contact surface to facilitate the full extension of the pelvis and prevent pelvic rotation, pelvic obliquity and unwanted hip flexion.

Pelvic Support Board Upper Back Pad

Used in conjunction with the **Pelvic Support Board**, the back pad offers support for the child's upper back. A head support can also be mounted to the back pad to help keep a neutral head position for those children who hyper extend their neck and head when standing in prone.

Mini Kit

Developed to facilitate standing for smaller children, helping to improve muscle development and weight-bearing through the feet. The **Mini Kit** can be removed once the child has grown sufficiently to allow the Standz unit to be used in its conventional format.

There may be limitations on the range of leg positioning options given the small stature of the child and we recommend that an assessment is carried out to identify the suitability of this accessory before purchase.

Knee Blocks

Innovative and flexible PU **Knee Blocks**; adjustable for height, width, angle, and depth. Available in 2 sizes, making them suitable for smaller children and come complete with padded, de-rotational knee straps.

In addition to the base you will need to add the following accessories to ensure a minimum safe configuration.

Prone – sandals, knee support, pelvic support & trunk support.

Supine – sandals, knee support, knee support straps, pelvic support, trunk support & head support.

Multi-Axis Knee Supports

Use for supine standing only. For use where there is the need to accommodate fixed knee flexion.

Standard Knee Support

Individual knee supports offering the ability to accommodate knee flexion when used in supine (requires **Knee Straps** when used in supine). Width adjustable to correctly gutter the knee in position.

Standard Knee Strap

Intended for use in conjunction with the **Standard Knee Supports**, the **Standard Knee Strap** simply holds the knee in the desired position and prevents knee flexion from occurring.

Rotation Control Knee Straps

Intended for use in conjunction with the **Standard Knee Supports** the **Rotation Control Knee Straps** allow for a cuffing of the child's leg and allow for control of internal or external knee rotation.

Arched Foam Pad

A simple foam pad that is shaped to fit behind the knee to ensure a comfortable standing position when accommodating knee flexion.

Sandals

Available for Standz in either size 1 or 2, these **Sandals** ensure solid foot placement to give the foundation for effective leg positioning and safe standing. Also available with **Ankle Snugs**.

Tray Mounting Bracket

Required to mount the **Multi-position Tray** – simply slots onto the Standz base and is adjusted using simple, comfortable wing knobs. Allows for the tray to be quickly changed from supine to prone or vice-versa, with no need to remove the mounting bracket.

Multi-position Tray

Cleverly designed to be used in both prone and supine, the tray is angle adjustable to ensure the correct position in either application.

Prone Only Tray

Shorter support arms allow for unobstructed movement of legs into the abducted position.

Tray Extension Wings

Strongly recommended for use in supine and just as effective when used in prone, these wings attach to the rear of the tray to offer a support surface for the child's elbows and prevents arms from slipping off the tray.

Tray Holder

A clever addition that offers a solution of where to store the tray when it is not being used.

Anti-tamper Conversion Kit

Allows for all wing knob adjustments to be converted to a tool fixing – ideal for environments where other children may tamper with the original wing knobs.



Standz by Jenx Technical Specifications

	Measurement	Units	Prone	Supine
Age Range	approx.	yrs	1 - 9	1 - 9
User Weight	max	kg	45	45
User Height	max	mm	1380	1380
Angle Range (support surface angle range)	min - max	degrees	5° - 90°	5° - 90°
Chest Width	min - max	mm	175 - 305	175 - 305
Hip Width	min - max	mm	175 - 305	175 - 305
Kneeblock Width (centre to centre)	min - max	mm	130 - 210	130 - 210
Leg Abduction	min - max	degrees	0° - 30°	0° - 30°
Footplate Angle	dorsiflexion - plantar flexion	degrees	10° - 25°	25° - 10°
Height of Stander (floor to top of upper pad)	in upright	mm	1080	1080
Height of Stander (floor to support board)	at horizontal	mm	810	810
Height from footplate to top of chestplate	min - max	mm	580 - 1000	580 - 1000
Base Footprint	width x length	mm	645 x 890	645 x 890

Find Your Distributor

To locate one of our
carefully selected
distributors near you, visit:
jenx.com/where-to-buy



@JenxLtd



@JenxLtd



@JenxLtd



Jenx Limited, Wardsend Road, Sheffield S6 1RQ, United Kingdom
tel: +44 (0) 114 285 3376
email: export@jenx.com
www.jenx.com

JENX® 
Positioning for life