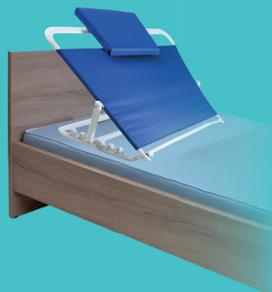
- ▶ Rückenstütze
- ▶ Backrest

- ▶ Gebrauchsanweisung
- ▶ User manual

Technical specifications:

Item no. 46 500 / Backrest					
Material:	frame powder coated steel, surface backrest: coated nylon, position holder: plastic	Dimensions:	approx. 50 x 65 x 32,5 / 48 cm	Adjustments:	seat back angle five times adjustable
		Width supporting surface:	approx. 60 cm		
Colour:	white / blue	Lifespan:	approx. 10 years	Max. load:	113 kg

Cleaning / Disinfection

- ▶ When cleaning do not use any sharp, abrasive object or cleaning supplies.
- ▶ All parts are best cleaned with a damp cloth under using mild household cleaners.
- ▶ To care for the plastic surfaces specially plastic suitable products can be used.
- ▶ The backrest can be disinfected with a commercial, chlorine-free disinfectant.
- ▶ **Important:** When using disinfectants, the instructions for use of the manufacturer must be observed.

Warranty

According to the statutory warranty claims. Excluded from the guarantee are defects resulting out of natural wear and tear, intent, negligence or improper handling or use and defects which affect the value or usability of the tool only insignificantly.

We reserve the right of modifications serving the technical improvement.

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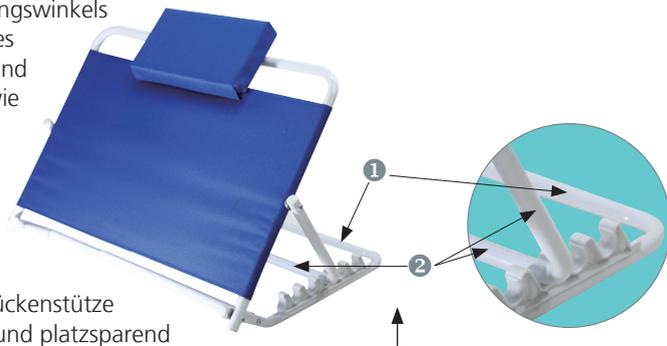
EN**Intended use:**

The backrest allows comfortable sitting and lying in bed. Especially in case of illness, when staying in bed for a longer time is required and there is no adjustable headrest, it can have a relaxing effect. Different positions between lying and sitting can be selected by 5 adjustable heights. Without muscular strength you can hold the necessary, preferably upright sitting / lying position e.g. to take food or to drink. A pillow fixed with Velcro can cushion the head additionally.

Before first use

Inspect all parts for shipping damage. Defective items must not be used (any longer), please contact your dealer.

Assembly

- ▶ The backrest is delivered preassembled and has to be positioned on the mattress.
- ▶ Unfold the backrest and put it in bed, so that the lower spar ① with the rounded corners ends at the headrest.
- ▶ The required position can be adjusted with the bracket ②. Therefore you have to bring the bracket in the desired position. Pull it down into the recesses of the position holder with soft pressure.
- ▶ To change the tilt angle the clamp is released by slightly pulling up, then in another position down again.
- ▶ For additional padding the pillow can be placed on the backrest too. After use the backrest can be folded and put away.

Usage

- ▶ Before each use please check, if the clamp is surely locked in the recesses of the position holder.
- ▶ The backrest must end at the headrest or the wall in order to avoid slippery.
- ▶ Lean back on the backrest centrally (max. load 113 kg).