



SimplyWorks® Range

The world's only fully integrated range of wireless Assistive Technology devices.

The range is huge– this is just some of it!



What does it do?

- ▶ It allows access to computers and tablets, toy control, mains appliance control, AAC and much more!
- ▶ It is very easy to set up– as the name suggests!
- ▶ It can be configured for a single user, or for multi-user cooperation.
- ▶ It's as flexible as your imagination.

Computer Access

- ▶ SimplyWorks® Receive and Receive:2 plug directly into a USB port.
- ▶ Works with PCs, Macs and Chromebooks.
- ▶ They are functionally the same– it's just a choice of format.



Did you know–
No drivers are
required– ideal
for managed
computer
suites used in
many schools.

- ▶ Each supports up to 6 transmitters.
- ▶ It doesn't matter what the transmitters are– any 6 in any order. For example....

Simple Switch Access

Did you know–
Receive:2 and
two 75mm
wireless
switches are
available as a
cost effective
starter kit?

- ▶ **Receive** (or **Receive:2**) and up to 6 switches.
- ▶ Two switches is a very common configuration for switch adapted software– scan and select.



- ▶ Switch function is set using a simple display on the bottom of each switch.
 - ▶ Choose from mouse, keyboard and gamepad switch functions.

Cursor and Keyboard

- ▶ If a user has the cognitive skill to use the cursor...



Choose
from
Trackball
or Joystick



e.g. Left
click



e.g. Right
click or
Draglock

- ▶ Perhaps add a keyboard.....




Did you know– the
keyboard is made
under licence by
Clevy and is
available either from
Clevy or Pretorian.

Multi-user

- ▶ Fact: Interactive Whiteboards are not that interactive!
- ▶ Make your classroom more interactive with a multi-user configuration of SimplyWorks....

Or use 6 switches..
..or 6 keyboards..
..or a combination–
the choice is yours!

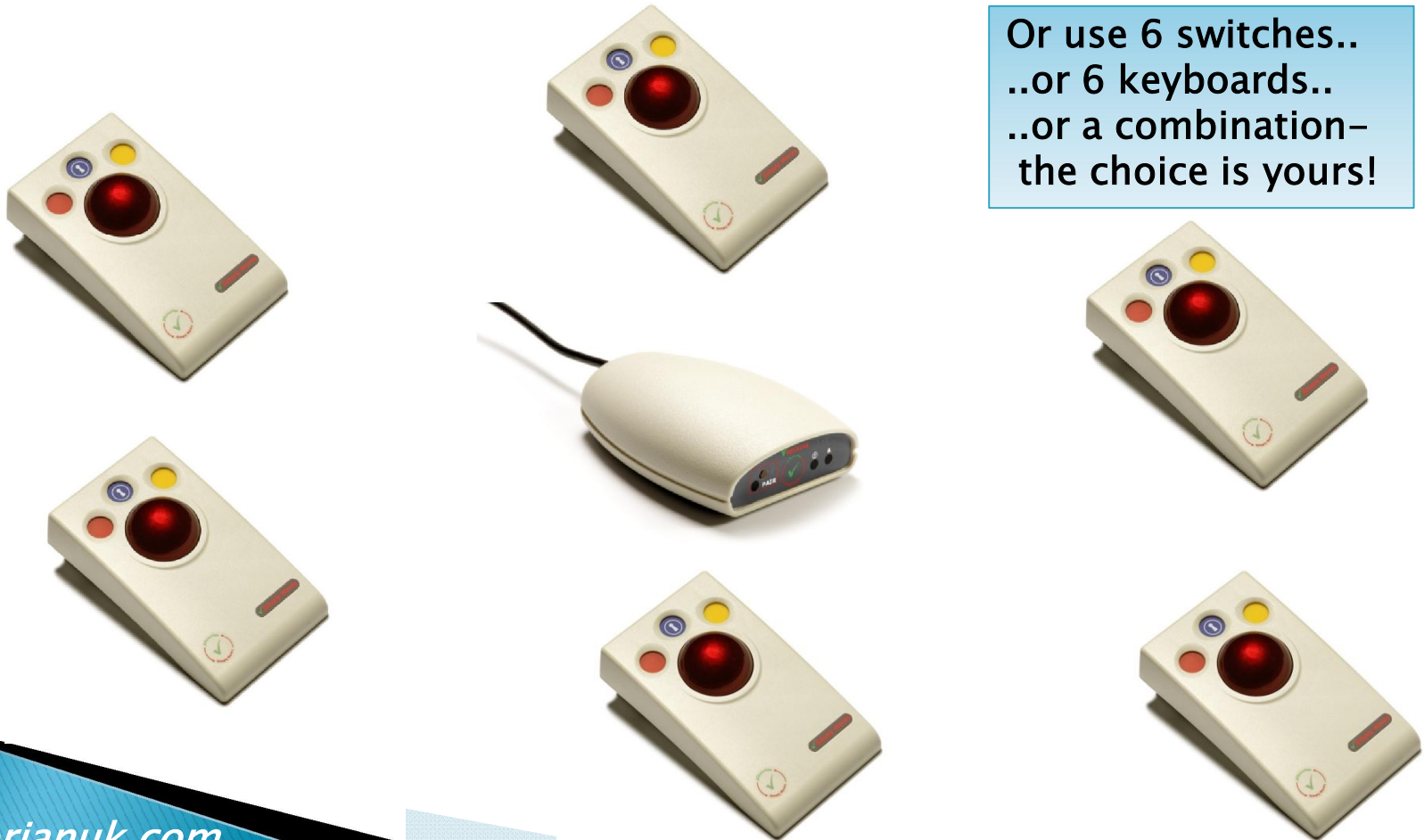

Alternative switches.

- ▶ Pretorian produces SimplyWorks switches in 125mm (5") and 75mm (3") diameter.
- ▶ If you need to use other types of switch with SimplyWorks– use Send.
- ▶ You can plug any switch into Send– for example squeeze switches, pillow switches, sip/puff etc...



What about Tablet Computers?

- ▶ No problem!
- ▶ For Android use **Receive Micro**. Just the same as Receive but with a micro-USB plug suitable for Android devices. Plug it in and a cursor appears, giving you access to all the SimplyWorks range on a tablet!*



Apple as well?

- ▶ Yes!
- ▶ SimplyWorks for iPad is a 'bridge' between SimplyWorks and iOS.
- ▶ It's a Bluetooth device that supports switch adapted app access and iOS Switch Control via switches or joystick...
- ▶ ...and the keyboard works too!



All your
SimplyWorks
kit



Note: We only recommend using the SimplyWorks for iPad if you have lots of existing SimplyWorks kit that you want to use with an iPad. If not, please consider Pretorian's dedicated iPad products.

Toy Control

Control
Pro



- ▶ You can wirelessly control toys and other battery powered equipment with SimplyWorks
- ▶ Use either Control (1 output) or Control Pro (2 outputs)...
- ▶ Choose from Direct, Latched or Timed functions
- ▶ Control Pro also includes cooperative features.



Control




Did you know– if you pair a Joystick with Control or Control Pro you can use the stick as a ‘wobble switch’? Very useful for users who find switches difficult.

Toy Control continued...

- ▶ Excitim makes a range of SimplyWorks toys (under license from Pretorian) with Control built into them– a truly wireless solution!



www.excitim.com

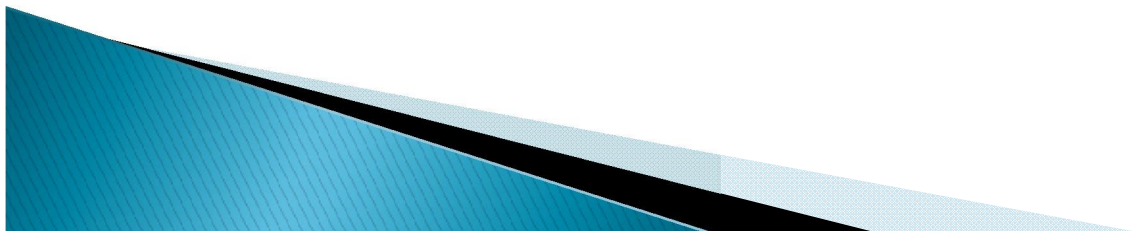

Appliance Control/ Environmental Control

- ▶ Use Energise to control mains appliances.
- ▶ Anything from lamps in a domestic setting through to food mixers for inclusive classroom activities...
- ▶ Choose from Direct, Latched or Timed functions.
- ▶ Available with EU, UK and (coming soon) US sockets.



Did you know– by using wireless switches, the user is afforded the ultimate safety as there is no physical connection between the switch and mains appliance. Makes your classroom risk analysis easy!

AAC

- ▶ Use Smooth Talker with SimplyWorks switches– unlocks some of the unique features built into Smooth Talker such as Converse and Auditory Prompt modes.
- ▶ You can also pair Smooth Talker with Control or Energise to give ‘rewards’ during particular messages....



AAC continued...



Mixing up your ingredients!



AAC continued...

- ▶ ...you can even pair it with Receive to make something happen on a computer during a particular message.



Watch the
sharks
swimming!

Other Products

- ▶ We have only presented some of the vast SimplyWorks range in this presentation, and we're continually adding new ones...
- ▶ ...like our new Kinetic which is a kinesthetic learning device.



Wrap Up

- ▶ SimplyWorks is all about flexibility and ease of use.
- ▶ We hope this presentation has given you some ideas on how best to use the range to create new and fun activities with good learning outcomes.
- ▶ Questions? We love to receive them! Email pretorian@pretorianuk.com and we'll get back to you within 24 hours.
- ▶ *Thanks for watching!*