

Wheeled Commodes

Toileting



Maximum user weight

160 kg **25 st**



1 Atlantic Commode Shower Chair Days

The Atlantic Commode and Shower Chairs are stylish, easily manoeuvrable and very easy to clean. Available in three seat widths: 455mm (18"), 505mm (20") and 560mm (22") to accommodate a wider variety of users. The weight capacity of 18" model is 160kg (25st) and 225kg (35¼st) for the 20 and 22" models. The chairs are manufactured from durable plastic coated steel and are ideal where infection control is an issue. All upholstery and braked castors are colour co-ordinated in blue. The multi function armrests are: swing back; drop down; detachable and come complete with an easy to operate hygienic arm catch. The hygienic white clip-on apertured seat, backrest and arms can easily be removed for cleaning or stored away. A toilet roll holder is supplied with all models.

Supplied with a square bowl as standard, however, an optional perfection pan rack can be fitted enabling the chairs to be used with a re-usable bed pan and a disposable bed pan. Models 599DPR and 599FDPR are supplied with a disposable pan rack as standard, enabling the use of a disposable pan only. For full specification, please see the reference table on the adjacent page.

599	455mm (18") Without footrests
599F	455mm (18") With footrests
599DPR	455mm (18") Without footrests, with disposable panrack
599FDPR	455mm (18") With footrests & disposable panrack

Accessories and Spares

SA459B	White Aperture Seat for 599 & 599F
SA459A	White Aperture Seat for 599DPR & 599FDPR
SA562A	White Aperture Seat for 598, 598X, 598F & 598XF
ACC2	Perfection Pan Rack
AA2643W	Perfection Bed Pan
PL512	Standard Square Bowl
09 128 2904	Disposable Bed Pan



Wheeled Commodes

Toileting



1 Atlantic Bariatric Commode Shower Chair Days

The bariatric versions have the added feature of a front cut-out on the white aperture seat to enable easier personal cleansing.

- 598** 505mm (20")
Without footrests
- 598F** 505mm (20")
With footrests
- 598X** 560mm (22")
Without footrests
- 598XF** 560mm (22")
With footrests

Maximum user weight **225 kg** **35 1/4 st**



Specifications	599 Series	598 Series	598X Series
Overall Height	995mm (39 1/4")	995mm (39 1/4")	995mm (39 1/4")
Overall Width	600mm (23 1/2")	620mm (24 1/2")	670mm (26 1/2")
Overall Depth	780mm (31")	780mm (31")	780mm (31")
Seat Height - Padded	545mm (21 1/2")	535mm (20 3/4")	535mm (20 3/4")
Seat Height - Aperture	535mm (20 3/4")	505mm (19 3/4")	505mm (19 3/4")
Clearance	*See below	440mm (17 1/2")	440mm (17 1/2")
Seat Width	430mm (17")	510mm (20")	560mm (22")
Seat Depth	460mm (17 3/4")	430mm (17")	430mm (17")
Castors	130mm (5")	130mm (5")	130mm (5")
Maximum User Weight	160kg / 25st	225kg / 35 1/4st	225kg / 35 1/4st

*Clearance 599 Series (Square Bowl) - 480mm (18 3/4"), (Disposable Pan Rack) - 470mm (18 1/2")

COMMODE CLEANING

Atlantic: 598, 598X & 599 and Atlantic Wave: 84618/20/22

COMMODE CLEANING

Atlantic: 598, 598X & 599 and Atlantic Wave: 84618/20/22

ALL COMMODES SHOULD BE CLEANED AFTER USE OR CONTACT WITH PATIENTS.

Depending upon the severity of contamination, several methods of cleaning are available.

Remove any faecal matter and urine with soap and water.

To Clean only:-

Use general purpose neutral detergent and warm water. The chair may be showered down to assist cleaning.

To Disinfect:

To disinfect, clean as above and wipe over with a solution of hypochlorite, rinse and dry. Do not use harsh abrasives or chemicals.

Follow the instructions on preparing and handling disinfection solutions.

When using wipes Days Healthcare recommend that you use Steri-X Wipes with advanced barrier control.

The Atlantic and Atlantic Waves are suitable for cleaning using hydrogen peroxide solution.

The Model 599 Atlantic (fixed back) and all Atlantic Waves can be steam cleaned.

The Atlantic models 598 & 598X series -(detachable backs) cannot be steam cleaned.

Version 1 -0209

GUIDE: ATLANTIC COMMODE CLEANING, USING WIPES GUIDE: ATLANTIC COMMODE CLEANING, USING WIPES

Wear gloves and apron

USING A NEW WIPE

Wipe pushing handle. Wipe backrest and armrests (can be detached for thorough cleaning).

Remove polyurethane overseat pad and wipe top and bottom.

Dispose of Wipe in bin.

USING A NEW WIPE

Wipe top of white plastic aperture seat (can be detached for thorough cleaning).

Dispose of Wipe in bin.

USING A NEW WIPE

Wipe underneath of white plastic aperture seat, including rack.

Dispose of Wipe in bin.

USING A NEW WIPE

Wipe frame. Wipe footrest (plastic footplate can be detached for thorough cleaning).

Dispose of Wipe, Apron & Gloves in bin.

DATE & SIGN TAPE AND APPLY TO COMMDES.

Wash and gell hands.

Days Healthcare U.K. Ltd. North Road, Bridgend Industrial Estate, Bridgend CF31 3TP

(t): 01656-664700 (f): 01656-664750 (e): info@dayshealthcare.com (w): www.dayshealthcare.com