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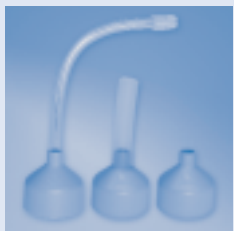
Contents

Foreword	15
Sources	15
Function	16
Placement	16
3-in-1 Oral Adapter	18
Care	19
Articulation	20
Individual Settings	21
Volume	21
Pitch ('Speech Melody')	22
Two Screw Caps	23
Points to remember	24
More SERVOX products	25
SERVOX Collection	25
Dear Customer	26

Foreword

After a laryngectomy, one of the principal aims of therapy is to restore the ability to communicate with the voice as soon as possible.

With the Servox Digital speech aid we offer you a tool which in the recent decades has proved itself through worldwide success.

Confident handling of the device, plus a knowledge of the various settings and their optimal usage is important for successful usage of the device, and a good level of clarity.

A referral by a specialist (for example, a speech therapist) is desirable in this regard. If this is not always possible, this brochure serves to introduce to you the various setting and usage options for the SERVOX® *digital*, as well as the basic principles of use.

We hope that with this added assistance you will be able to customise the SERVOX® *digital* so that it meets your individual needs as best as possible. The two screw caps can also help you with this, as they provide you with the choice between a soft or hard tone.

If further questions should arise concerning the use of the SERVOX® *digital* speech aid, we are happy to provide you with help and advice.

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Function

The SERVOX® *digital* speech aid is battery operated. When Button A or B is pressed, a magnetic coil with a plunger starts to vibrate and beats against the hard membrane of the screw cap. The resulting vibrations pass through the tissue of your neck or the soft tissue under your chin, and into your mouth. The air in your mouth starts vibrating in turn, and in this way a whisper can be turned into voiced speech.

Placement

When you first receive your new speech aid, it is important to find a suitable place to hold it against.

To achieve the best possible sound quality, there are three different areas of placement:

- on the side of the neck
- under the chin
- on the cheek

To find the best spot for you, first practise in front of the mirror, or with a relative or therapist who can help you handle the device and correct you if necessary.

Try out your speech aid on each of the areas mentioned above until you find your 'best spot'; the spot which allows the tone to come through the best.

Position the speech aid so that the tone sounds clean and no unwanted distortion (buzzing) is heard. Whilst doing this, open the mouth slightly or say a prolonged vowel, for example a long "Aaaaaaa" sound.

You can experiment holding the speech aid with varying pressure against the skin.

Take note that creases or wrinkles in the skin, scars, facial hair and also larynx protection bibs affect the quality of the tone.

Once you have found a good position, repeat finding that place until you can find it again confidently. Use your non-dominant hand to hold the speech aid, otherwise you will find yourself constantly swapping over to shake hands or write.

If the vowels you are pronouncing are clear, you can move onto words. When practising words and phrases, don't move your SERVOX® *digital* away from its position, for example whilst turning your head or moving your jaw. Make a concerted effort to maintain contact with the skin, as a loss of contact will affect the quality of the tone produced and therefore have a negative effect on your clarity.

Ensure when beginning and ending speech that you start and stop the tone in correct synchronisation through use of the buttons.

Don't speak too quickly at first, take pauses where it makes sense, so that your speech sounds as natural as possible.

3-in-1 Oral Adapter

If due to tissue swelling on the throat or cheek after the operation and/or radiation therapy, you do not manage to find a suitable place to place the speech aid, the 3-in-1 Oral Adapter can be used.

There is a choice of three different possibilities:

A Long Tubed Adapter

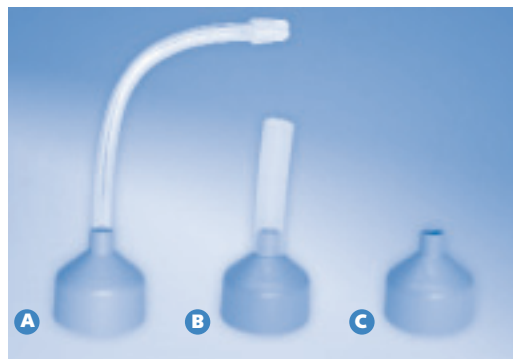
- Easy to hold at a comfortable height
- Tube bends to fit
- Discrete
- Hygienic – saliva cannot easily enter the adapter

B Short Tubed Adapter

- Made from a soft plastic, therefore more comfortable in the mouth
- Hygienic
- Louder than option A
- Easy to clean and to use

C Tubeless Adapter

- Highest volume
- Closer to the mouth

Place the appropriate tube in the mouth in such a way that it does not impede the free movement of the tongue, lips and jaw.

The tip of the long tubed adapter should be placed between the upper row of teeth and the inside of the cheek.

With the short tubed adapter, a small section should slide sideways into the mouth. The tubeless adapter should be placed in the corner of the mouth.

In this way a keynote reaches the mouth, where in combination with correct articulation, it can be transformed into voiced speech.

To a certain extent each tube presents a hinderance to speech, therefore it is very important to articulate your words well in order to maintain clear speech.

Practice inserting and removing the tube until you consistently find the best position for you. At first a mirror may be useful to aid you in doing this.

Care

Regular cleaning of your oral adapter is essential. We recommend warm water or slightly soapy water. From time to time the tubes should be replaced by new ones.

Articulation

As mentioned previously, precise articulation is vital to achieve clear speech. It helps to practise beforehand a whisper only using the air in your mouth. There are various „pseudowhisper“ sounds which can be formed just using the air in your mouth, that don't require the voice.

Practise the sounds listed below without your speech aid, so that you can feel the puff of air from the mouth and hear the sounds themselves:

Plosives:	p – t – k
Fricatives:	f – s – sh – th – h
Linked consonants:	ps – ts – ks – ch – st

Once you have perfected the pronunciation of single phonemes, you are ready to try words.

Practice words such as:

Past, Toast, Cook, Fast, Tool, Ship, Cheese, Lose, Box, Stool

When practicing these sounds and words, ensure that no accompanying breathing sounds can be heard from the stoma. This reduces clarity, as the breathing sounds can overpower the voice.

Think about whispering softly, this alone will help to reduce the breathing sounds. You have spent many years speaking with the air you breathe out, now it's just a question of changing habits and only using the air in your mouth to articulate.

At the same time though, ensure that you do not over-articulate or use your speech apparatus (jaw, lips, tongue) excessively as this can result in a throaty voice. It sounds then as if you are speaking right from the back of your mouth and is difficult for listeners to understand.

Individual Settings

Volume

The individual volume setting on your speech aid is very important for enabling your voice to fit to any particular situation.

- The 'soft' volume level is for use in situations such as speaking on the phone, dining at a restaurant or in places such as churches or libraries where a quieter voice is required.
- 'Medium' is the everyday volume level, used for general conversation in environments such as the home.
- The 'loud' volume level should be used only occasionally in situations where you feel it to be appropriate.

The volume can be altered for both buttons, together or separately. Push the rocker switch in an upwards direction to raise the volume for both buttons, and downwards to lower the volume.

If you wish to alter the volume for just one button, press this button down whilst pushing the rocker switch in the desired direction.

Pitch ('Speech Melody')

To begin with, seek assistance from a speech therapist to help you set the desired pitch. Please also follow the advice in the instruction manual. This is achieved using the dip-switch inside the speech aid. Relatives who knew your original voice well should be able to help you choose the pitch that best matches that.

The difference in pitch between buttons A and B is usually a halftone, this gives speech a more natural sound which we can call the 'speech melody'. You should learn to change quickly between the two buttons in order to add emphasis to words and phrases where it is naturally required.

Examples:

Hi , how are you?	Yes , I like that!
Fancy a coffee ?	What's your name?

In these cases, Button B should be used for the word or syllable in bold print, as this is where the emphasis lies. This button can be operated either by a rocking movement using the thumb, or by using your index (pressing button A) and middle finger (pressing button B).

When you first begin speaking with your SERVOX® *digital* you will probably already have enough to do with coordinating your breathing, articulation and speech. However, having already worked on this first step and now feeling more comfortable with the speech aid, you can begin to introduce your own speech melody. Exercises such as reading poems or other texts out loud will help to give your speech more variation and expression.

In addition, it is possible to alter your sound by moving the speech aid either slightly above or below your usual spot, and varying the pressure with which you hold it against the skin.

Try to maintain your natural facial expressions and gestures during speech.

Two Screw Caps

The individual character of the voice obtained through the use of an electronic speech aid is partly limited by design factors.

Good articulation, facial expressions and gestures, as well as the coordinated and confident usage of the buttons are crucial for clear speech.

With the SERVOX® *digital* speech aid we provide you with two screw caps; one with a harder tone, the other with a softer tone.

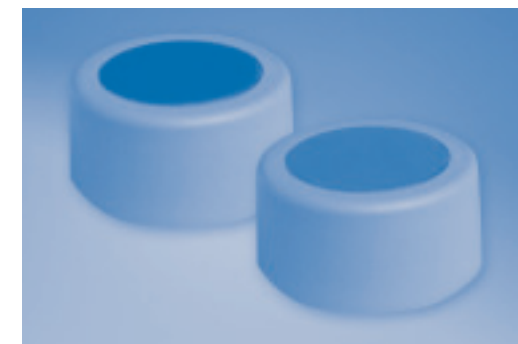
In this way we are offering you a greater possibility of the speech aid settings suiting your personal needs and expectations.

Please try out both screw caps once you are familiar with the principles of using the device, and you are confident in using it. If need be, let other family members help you decide which screw cap best and most clearly reproduces your original voice.

Please note:

- Screw cap with dark blue hard membrane = harder tone
- Screw cap with mid-blue hard membrane = softer tone

When changing the screw cap, please ensure that the O ring is not squashed.

Points to remember

Posture: To be as successful as possible with your SERVVOX® *digital*, simply stay relaxed and loose, particularly around the neck and shoulder area.

Placement: Press the speech aid lightly against your 'best spot' (at the side of your neck, under the chin or on the cheek) to achieve a clear, buzz-free sound. Creases or wrinkles in the skin, scars, facial hair and protection bibs affect the quality of speech.

Screw cap: If the cap is screwed onto the device too tightly, the clarity of the tone will be affected.

Pitch: Match the pitch of the speech aid to that of the voice before surgery, ask relatives for help with this.

Synchronising tone and speech: Don't swallow the beginning and end of words; press button A or B immediately at the start of speech and release as soon as speech is completed.

Breathing: Avoid sounds from the stoma by practicing breathing exercises without your SERVVOX® *digital*, for example, blowing a whistle or blowing out a candle. Use only the air in your mouth for this. Try dividing your breathing into stages: Inhalation – Exhalation – Pause.

Articulation: Speak clearly without swallowing any syllables. Don't speak in a continuous tone, but avoid overarticulating.

Speech tempo: Speak smoothly with regular pauses.

Expression: Enhance your expression by incorporating your natural facial expressions and gestures.

Note:

If you are using an electronic speech aid and at the same time learning oesophageal speech, take care not to mix the two together. When using the SERVVOX® digital the sound is produced by the speech aid.

In oesophageal voice you are using your body's own voice, and air from the mouth must firstly be moved into the oesophagus in order to voice your speech. Be sure that you don't use both methods together, which means: don't speak with the oesophageal voice at the same time as using your speech aid. When you use the speech aid, just use your pseudowhisper.

More SERVVOX products

SERVVOX Collection

These products protect the tracheostoma from dirt, dust and the cold and discreetly cover the area from view.

Larynx protection bib

- Velcro or plastic clip fastening
- Available in four different designs and in three colours

Larynx neckerchief

- Diolene tulle filter
- Velcro on one side to allow for easy fastening when movement is restricted

Larynx scarf

- Diolene tulle filter
- Adds elegance to the neckline of blouses and shirts

Larynx rolli

- Diolene tulle filter
- Concealed Velcro fastening

Larynx rolli shirt style with zipper

- Diolene tulle filter
- Can be easily pulled on over the head
- Zipper in the side seam

Tracheofix

- Guarantees optimal filtering and moistening of the air
- Can be attached over the tracheostoma with an adhesive strip

Please ask your distributor for details.

Dear Customer

We are pleased to offer our descriptive CD ROM to further assist you. The CD ROM with animated pictures enhances the written explanations as part of the operating instructions. Through simply watching the new animated pictures, you are able to see and understand how the device is placed.

This CD provides you with the following information:

- Medical background
- Operating the SERVVOX® *digital*
- Operating the Charger LA 2.40
- First steps for using the SERVVOX® *digital*

Please contact our local distributor to order this CD.
Please refer to **article number 18068** when ordering.