


Ready To Speak

Short instructions for the **SERVOX® digital**

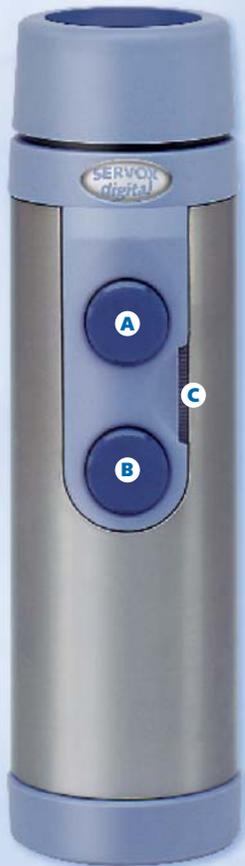

Adjusting the Volume

The volume can be adjusted using the volume controller (rocker switch). The following modes are possible:

a) Setting buttons A and B with different volume levels.

To adjust the volume of each button separately: first push the button (a tone is audible), then either increase the volume by pushing the rocker switch upwards at the same time, or lower the volume by pushing it downwards.

b) Changing the volume for both buttons A and B

No button is pressed in this case (no tone is heard). Increase the volume by pushing the rocker switch upwards or decrease the volume by pushing it downwards. Press the corresponding button to check that the volume has been set accordingly.

- A** Button A
- B** Button B
- C** Volume Controller (Rocker Switch)

Please consult the **SERVOX® digital** instruction manual for further information.

Our local distributor:

SERVOX AG

Biberweg 24-26 · 53842 Troisdorf · GERMANY
P.O. Box 31 09 · 53831 Troisdorf · GERMANY
Tel.: +49 (0)22 41/93 22-0
Fax: +49 (0)22 41/93 22-277
e-mail: info@servox.com · Internet: www.servox.com



Ready To Speak



with
SERVOX® digital



Ready To Speak

Do you know a longer lasting speech device?

"After 2 1/2 years of using my SERVOX speech aid it has never let me down. I was able to use



the batteries, included in the set for 2 years. Although I bought a Tru-Tone it stays in the car as back up." *Jimmy LaFrance*

"I have been using a Servox for 6 years. I started with the Nu-vois electro larynx after surgery 7 years ago, but I was very unhappy with it. I switched to the Servox and have been using it ever since." *Norm Edelman*

"I am 60 years old and was diagnosed with larynx cancer in 1998. On 17th August this year, it will have been 5 years since I had my voice removed. I was using a Servox 2 days after my operation. In 2000 I was also diagnosed with breast cancer. I have been told by the doctor that they have caught all the cancer. I'm voiceless but not speechless, and I am now able to share my experience as a speaker for the Rogers Lowell anti-tobacco Collation and Fayetteville anti-tobacco Collation." *Joy Ewalt*

"My name is Doris Gifford. In May 1998 I had a laryngectomy. 3 weeks after being released I was given a Servox speech aid. I have found the art of using a Servox. There are times when I wish for my God given voice, but I am alive and talking and therefore very grateful for my Servox."

68 year old Richard Jacobs has used the Servox since 1994 and it is the only speech tool he has ever used. "It is the easiest and best way to communicate with my friends



and family. Everyone I talk to says I am really good with the Servox but I think it's the unit."

"To me the Servox is my life line. I have been using one as my only means of communication since April 1996 and very comfortable with using it. I picked up the technique of using the device 2 weeks after surgery and most people can understand me without any trouble." *Joy Scott*

"I took my Servox home from the hospital straight after my surgery. Could use it immediately with an oral adapter, but not as clearly as when I could place it against my neck, about 2 weeks later. I have never had any trouble and I still have my Servox as back-up since I got a TEP." *Roger Jordan*

"When I first had my surgery, I was handed a Servox and told use it. That was back in 1995. Since then I have fished it out from under the car with a fishing pole, dried it after the cat pulled it into the water bowl. I took it to nursing school, made it through, and graduated". Pam Ridgeway explains what she has been up to with her Servox.

Judy Rambolt cannot say how much she loves the Servox. "After almost 9 years the instrument is just another part of me". She has used both the Inton and the digital and explains that "they have both survived the major drop tests."



with SERVOX® digital



We care about your communication!

80 % of people can use the speech aid straight from the packaging and find the settings are already tailor-made for their needs.



- Digital Electronics gives you clearer sound and is more reliable than other analog units.
- The digital electronics give you less distortion and noise and the result is optimal sound quality.



- New digital technology incorporated in the two programmable buttons makes it possible to combine two different volumes with an individual pitch for each one.



- Oral connector available upon request. This enables you to speak the day after surgery and is a big moralbooster.



- You can change the safety carrying cord yourself and you don't have to mail it to our national repair centre anymore.

- You get two rechargeable batteries, so you can rest assured that you are not cut off during speaking.

- Free, new, attractive, modern, dark blue, nylon carrying case can also be worn on a belt if you desire and a separate compartment can hold a spare battery so you never have to go without talking.

With the help of therapists and multi software the patients can adjust the device to the individual and desired settings.