

PRINT DATE: Saturday, 24 April 2021



► VIEW DATASHEET ON-LINE www.chinesport.com/go/AV3121A0113G/

AV3121A0113G STAND-UP 500

This assistive device enables the user to assume and maintain an extreme upright position, and to move indoor assisted by a caregiver or relative. Users who are able to transfer themselves from their wheelchair or other sitting position to the standing aid and to lift themselves up unaided can use this classic manual model. If efficient and constant assistance is available this standing aid is also suitable for weaker users who only use it few times during the day.

The end user can grip the long side handles and the tray edges to lift himself up. If the user is unable to bring himself to a standing position, then he must be helped by one or more caregivers depending on his body weight. Help is needed to close the buttock support (only few users are able to perform this operation by themselves). The long side handles can be helpful at first stage to support the user in the transferring onto the footplates, and then for guaranteeing a safe condition during the lifting.

A series of adjustments without tools ensure the aid can be adapted to the user's morphological characteristics. In particular, the main frame along with the knees, hand and trunk supports are easily adjustable to the user's height by means of a gas spring to facilitate and reduce the caregiver efforts.

For better stabilization at trunk and hips level the long side handles can be adjusted in width and integrated with padded elements as accessory. The device is fitted with a wide service tray adjustable in tilt which can also be padded (please see accessories).

As alternative the STAND UP 300 model offer a gentle push by gas spring under the special seat, that can integrate the manual action of the user. This mechanism safely supports the user, who can stop and rest at any intermediate position before completing the lifting instead of what happens with this manual version.

Dimensions: 103.5 x 75 x 98 ÷ 146 h cm. Weight: 64 kg. Max load: 140 kg. User height: 140 ÷ 200 cm.












NOTE


A demonstration video of the main performances, and in particular for the lifting and adjustment methods of the entire frame to the height of the end user, is available with reference to a previous version (AV111110/STAND UP 1110) corresponding to and replaced by this model.


Technical specifications and dimensions

Dimensions :	103.5 x 75 x 98 ÷ 146 h cm
Base accessibility :	63 cm (enlarged)
Front and rear wheels diameter :	Ø 7.5 cm (transfer)
Ground handling :	assisted
Type of handling :	manual
Structure height adjustment :	hydraulic
Servo-assisted verticalization :	absent
Autonomy of continuous use :	190 min
Internal / external base width :	63 - 75 cm
Capacity :	140 kg
Weight :	64 kg

Documentation available online

Manual - Manual de uso y mantenimiento Spanish		.PDF 1.9 MB
Manual - Manuale d'uso e manutenzione Italian		.PDF 1.91 MB
Manual - User and maintenance manual English		.PDF 2.03 MB
Manual - User and Maintenance Manual Italian, English		.PDF 2.57 MB
Manual - Manual de uso y mantenimiento Spanish		.PDF 1.87 MB
Manual - Benutzer- und Wartungshandbuch German		.PDF 2.88 MB
Manual - Bedienungsanleitung German		.PDF 2.88 MB
Various - Nomenclatore Tariffario Italian		.PDF 38.04 KB
Compliance statement - Declaration of conformity Italian, English		.PDF 100.1 KB
Compliance statement - Declaration of conformity Italian, English		.PDF 103.7 KB

 The document is ready for download in reserved area.

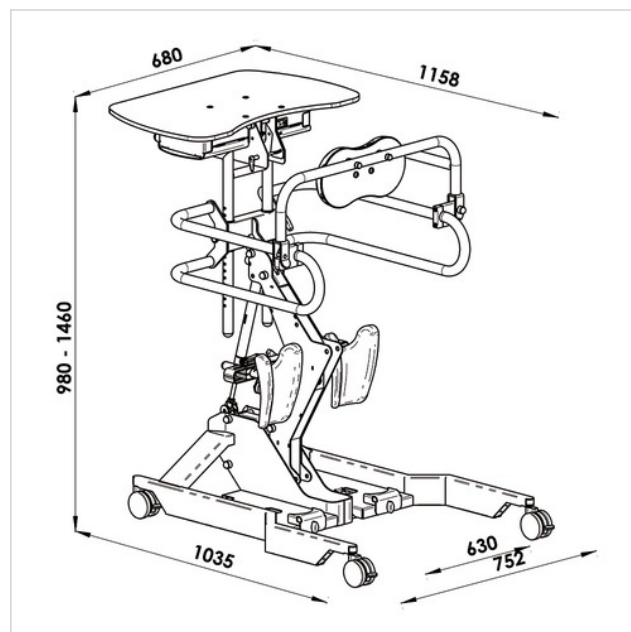

Accessories




AC0689 COLOURED PADDINGS

Made of soft foamed material with synthetic leather upholstery. The padding is applied to the specific metal structure. Available in three colours. Available in pairs.



AC0686 ADJUSTABLE HEEL RESTS

Back support and guidance for the feet, adjustable in width independently. Usable only with metal footplates. Supplied with foot strap for stabilizing the foot. Available in pairs.

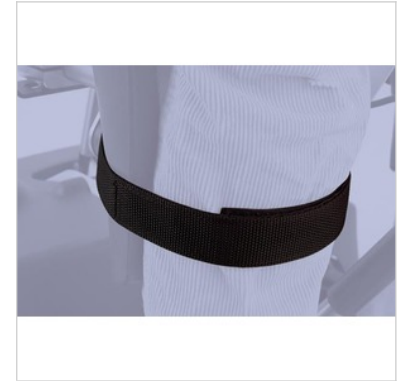


AC0014 LEG STRAP

Leg strap to stabilize the leg. Available in pairs.



AC0693 FRONT SUPPORT

This broad support is useful whenever there is not a proper control of the trunk by the user. Adjustable in depth in 7 positions: from 12 to 84 mm max.

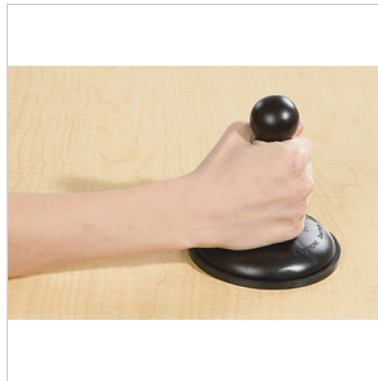

01608 HAND ANCHOR

This ergonomic anchor can be applied to any smooth surface to achieve greater stability and safe positioning of the hand.